Safety for life

OVERVIEW **FIRST AID KIT PREMIUM**

1x Tweezer

Intended use: For removing splinters or other small objects in an emergency situation.

Warning: Keep out of reach of children. Be cautious when using this device because of its slightly pointed tip.

1x Tick Remover

Intended use: For removal of ticks.

Instruction for use:

1. Press suppressed key
2. Create a clamp for the tick
3. Slowly rotate and remove tick

Warning: Follow instructions carefully. Make sure that the tick is completely removed. Watch out for any skin color changes and seek medical attention if a red ring appears in the area where the tick was sitting.

1x Scissors

Intended use: For cutting plaster, bandage, gauze or other small first aid devices. The scissors have a rounded cutting edge in order to protect the user from self inflicted injuries.

Warning: Keep out of reach of children.

2x Safety Pin

Intended use: For attachment of triangular bandages, first aid sling or other first aid bandages.

Warning: Keep out of reach of children. Be cautious when using this device because of its slightly pointed tip. Do not use if broken.

2x Disposable Gloves

Intended use: To protect against infections. Disposable. Latex-free.

Warning: Non sterile. Avoid direct contact with the wound.

6x Skin Cleansing Swab

Intended use: For cleaning the skin and the area around minor wounds and abrasions.

Instruction for use: Wipe carefully around the area to be cleaned.

Warning: External use only. Avoid any contact with the eyes, mucosa, genitals and injured skin. Contains 70% isopropyl alcohol. Do not use in case of allergies. Product to be used once only. Check the integrity of the individual sachet before use. In case of repeated discomfort, ask your doctor for advice. If swallowed, seek medical attention immediately and show the packaging. Use before the expiry date on the sachet. Store in a cool, dry place. After use, do not dispose of the compress in the environment.

10x Finger Plaster

Intended use: For use on minor cuts and abrasions. Protects minor wounds. Read instructions prior to use; retain for future reference.

Instruction for use: Clean wound and surrounding area and dry skin before applying the adhesive bandage. Remove adhesive bandage from protective wrapper. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Use immediately after opening protective wrapper. Wrapper is sealed with natural rubber latex. Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove adhesive bandage. Not suitable for children under 2 years old. In rare cases, a reaction to the adhesive bandage may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in the face. In both cases seek medical attention immediately. If you are diabetic consult your doctor before use.

20x Plaster

Intended use: For use on minor cuts and abrasions. Protects minor wounds. Read instructions prior to use; retain for future reference.

Instruction for use: Clean wound and surrounding area and dry skin before applying the adhesive bandage. Remove adhesive bandage from protective wrapper. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change

dressings every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Use immediately after opening protective wrapper. Wrapper is sealed with natural rubber latex. Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove adhesive bandage. Not suitable for children under 2 years old. In rare cases, a reaction to the adhesive bandage may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in the face. In both cases seek medical attention immediately. If you are diabetic consult your doctor before use.

20x Kids Plaster

Intended use: For use on minor cuts and abrasions. Protects minor wounds. Read instructions prior to use; retain for future reference.

Instruction for use: Clean wound and surrounding area and dry skin before applying the adhesive bandage. Remove adhesive bandage from protective wrapper. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Use immediately after opening protective wrapper. Wrapper is sealed with natural rubber latex. Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove adhesive bandage. Not suitable for children under 2 years old. In rare cases, a reaction to the adhesive bandage may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in the face. In both cases seek medical attention immediately. If you are diabetic consult your doctor before use.

10x PU-plaster

Intended use: Transparent PU plaster, composed of a pad on coated polyurethane film with acrylate adhesive and release top liner. For use on minor wounds or on IV catheter. Protects the wound from contamination, allowing the skin to breathe for enhanced healing. Read instructions prior to use; retain for future reference.

Instruction for use: Clean wound and surrounding area and dry skin before applying the plaster. Remove plaster from protective wrapper. Position pad over wound and press firmly on adhesive area to ensure adhesion. Then remove the release liner on the PU film. Change dressing every 24 hours or when wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Use immediately after opening protective wrapper. Wrapper is sealed with natural rubber latex. Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or

infected remove plaster. Not suitable for children under 2 years old. In rare cases, a reaction to the plaster may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in the face. In both cases seek medical attention. If you are diabetic consult your doctor before use.

5x Blister Plaster

Intended use: Protects the skin and prevents blisters from forming. Reduces friction on skin. Providing instant pain relief and optimum healing conditions for heel blisters. Read instructions prior to use; retain for future reference.

Instruction for use: Clean and dry the skin properly before use. Remove the lower protective film without touching the adhesive surface. Apply the dressing directly on the skin and carefully smooth the edges and press firmly for 5-10 seconds on adhesive area to ensure adhesion. Remove the dressing once it begins to detach itself, gently peeling it off the skin. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Use immediately after opening protective wrapper. Do not use on wounds. If skin becomes irritated remove plaster and seek medical attention. If you are diabetic consult your doctor before use.

1x Adhesive Tape

Intended use: For fixing the dressings, compress, cotton ball or other small medical pads.

Instruction for use: Apply the dressing / compress on clean and dry skin using the tape to keep it in place.

After use, please handle it according to medical waste. Do not apply tape too tight, this will influence the blood circulation and feel uncomfortable.

Warning: Do not apply the tape directly on the wound. External use only, keep away from children. If wound bleeds excessively or skin becomes irritated or infected remove the tape and seek medical attention.

6x NaCl Wipe

Intended use: For cleaning the skin and minor wounds and abrasions. Can also be used for cleaning hands before treating an injury. They contain a mild and natural salt solution, but are not antibacterial.

Instruction for use: Clean the wound from the centre outwards to avoid getting bacteria into the wound. Throw the swab away after use.

Warning: External use only. Single use only. Never use if the package is damaged. Check the integrity of the individual sachet before use. In case of repeated discomfort, ask your doctor for advice. If swallowed, seek medical attention immediately and show the packaging. Use before the expiry date on the sachet. Store in a cool, dry place. After use, do not dispose of the compress in the environment.

Note! Instead of wipes you can use soap and water or an antibacterial solution. Soap and water is recommended.

40x Plaster

Intended use: For use on minor cuts and abrasions. Protects minor wounds. Read instructions prior to use; retain for future reference.

Instruction for use: Clean wound and surrounding area and dry skin before applying the adhesive bandage. Remove adhesive bandage from protective wrapper. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Use immediately after opening protective wrapper. Wrapper is sealed with natural rubber latex. Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove adhesive bandage. Not suitable for children under 2 years old. In rare cases, a reaction to the adhesive bandage may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in the face. In both cases seek medical attention immediately. If you are diabetic consult your doctor before use.

2x Burn gel

Intended use: For first aid use on burns, scalds and sunburn. First aid emergency burn gel. Cools, soothes and relieves pain.

Note! Water is best on burn wounds, continue to rinse with water on burns. Burn gel should only be used on burns if there is no water available.

Instructions for use:

- Tear open sachet
- Apply gel liberally to affected area. Cover if necessary. Seek medical attention if pain persists.

Warning: Do not use on serious burns, mucous membranes or the eyes. For external use only. Discontinue use if irritation occurs. Do not store at extreme temperatures. Do not use if packaging has been opened or damaged.

2x Sterile wound Dressing

Intended use: Bandage with compress for protecting a clean and dry wound.

Instruction of use: Cleaning and drying the wound and the skin around the wound. Apply the compress on the wound and wrap the bandage around. Finish by tying the two ends together

Warning: Replace the dressing on a daily basis, or more often if necessary. Single use only. Check the integrity of the individual sachet before use. Store in a cool, dry place. Use before the expiry date on the individual sachet. Must not get wet.

1x Ice Pack

Intended use: For relieving pain and reducing swelling from sprains and soft tissue trauma. May relieve itchiness and swelling from insect bites. Can be used for cooling when placed on wrists and neck to prevent heat stroke.

Instruction for use:

See packaging.

1. Squeeze firmly to rupture and activate.
2. Shake to mix contents.
3. Apply to affected area with a piece of cloth between skin and cold pack.
4. Discard it after use.

Warning: For external use only. Adult supervision required. Do not puncture. If bag breaks and the fluid contacts skin or eyes, immediately flush area with large amounts of water. Do not swallow contents. If contents are accidentally swallowed, drink large amounts of water (not milk) and contact local Poison Control Centre. Cold therapy should not be used by individuals with circulatory problems, unless under the direct supervision by a physician.

1x Triangular Bandage

Intended Use: To immobilise and keep splints in place. Use this bandage as a sling to support an injured, sprained or broken arm.

Instruction for use: Unfold the bandage and attach around the neck as shown.

Warning: Product to be used once only. Use before the expiry date on the individual item. Check the integrity of the individual item before use.

8x Adhesive Bandage

Intended use: For use on minor cuts and abrasions. Protects minor wounds. Read instructions prior to use; retain for future reference.

Instruction for use: Clean wound and surrounding area and dry the skin before applying the plaster. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove plaster. Not suitable for children under 2 years old. In rare cases, a reaction to the adhesive bandage may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in the face. In both cases seek medical attention immediately. If you are diabetic consult your doctor before use.

3x Rolled Gauze Bandage

Intended use: To hold compresses or other medical dressings in place.

Instruction for use: Wrap the bandage around the area where compress/dressing placed. Do not apply excessive pressure. Place the end under the lower layers or use a safety pin.

Warning: Never apply the bandage directly onto injured skin. Avoid pressing too hard. In case of discomfort, release the bandage to reduce the pressure. Store in a cool, dry place.

6x Dry Swab in Paper film

Intended use: For cleaning and drying the wound and the skin around the wound. Protecting a clean and dry wound.

Instruction for use: Open the package and wipe gently in the area to be cleaned / dried. Or place the swab on the area that needs protection.

Warning: Product to be used once only. Check the integrity of the individual sachet before use. Store in a cool, dry place. Use before the expiry date on the individual sachet.

2x Eye Drops 15ml

Intended use: Sterile saline solution for emergency rinsing of a wound or flushing the eyes. Sterile Sodium Chloride 0.9%

Instruction for use: Twist the lid. Rinse the eye, carefully.

Warning: Not for injection. Not to be taken internally. Do not use if damaged. Once opened, discard any unused solution. Use before the expiry date. Risk of infection by reuse. Storage temperature 5-30°C.

1x Respiratory sheet

Intended use: For emergency situations during artificial respiration. Prevents direct contact with the injured person's mouth, nose, and face. Provides protection against contamination between the patient and the rescuer in an emergency and allows air to ventilate.

Instruction for use: Lift the chin upward and press back on the forehead. Place the mask with the (+) side facing up and the (o) side against the injured person's mouth. Perform mouth-to-mouth resuscitation.

Warning: Small children: Do not use the breathing mask on small children. For single use only.

1x First Aid Blanket

Intended use: To protect against heat and cold. The emergency blanket can provide protection against heat, cold and humidity in emergency situations.

Instructions for use: Emergency blanket to be used once only. Open the sachet and unfold the blanket as required.

Two sides with different functions: The golden side absorbs heat and keeps you warm: providing a defence against potential hypothermia, maintaining your body temperature, and protecting you from the wind and humidity. The silver reflective outer side keeps you cool: keeps you chilled (cars, tents, etc.).

Warning: Do not cover your head, danger of suffocating. Single use only. Do not use the emergency blanket during storms, near to incandescent substances or if using an electrical surgical device (e.g. DSA). Do not cut or create wholes. Store in a dry place.

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