Safety for life

OVERVIEW **FIRST AID KIT PLUS**

30x Plaster

Intended use: For use on minor cuts and abrasions. Protects minor wounds. Read instructions prior to use; retain for future reference.

Instruction for use: Clean wound and surrounding area and dry skin before applying the adhesive bandage. Remove adhesive bandage from protective wrapper. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Use immediately after opening protective wrapper. Wrapper is sealed with natural rubber latex. Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove adhesive bandage. Not suitable for children under 2 years old. In rare cases, a reaction to the adhesive bandage may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in the face. In both cases seek medical attention immediately. If you are diabetic consult your doctor before use.

4x Skin Cleansing Swab

Intended use: For cleaning the skin and the area around minor wounds and abrasions.

Instruction for use: Wipe carefully around the area to be cleaned.

Warning: External use only. Avoid any contact with the eyes, mucosa, genitals and injured skin. Contains 70% isopropyl alcohol. Do not use in case of allergies. Product to be used once only. Check the integrity of the individual sachet before use. In case of repeated discomfort, ask your doctor for advice. If swallowed, seek medical attention immediately and show the packaging. Use before the expiry date on the sachet. Store in a cool, dry place. After use, do not dispose of the compress in the environment.

1x Adhesive Tape

Intended use: For fixing the dressings, compress, cotton ball or other small medical pads.

Instruction for use: Apply the dressing / compress on clean and dry skin using the tape to keep it in place. After use, please handle it according to medical waste. Do not apply tape too tight, this will influence the blood circulation and feel uncomfortable.

Warning: Do not apply the tape directly on the wound. External use only, keep away from children. If wound bleeds excessively

or skin becomes irritated or infected remove the tape and seek medical attention.

4x NaCl Wipe

Intended use: For cleaning the skin and minor wounds and abrasions. Can also be used for cleaning hands before treating an injury. They contain a mild and natural salt solution, but are not antibacterial.

Instruction for use: Clean the wound from the centre outwards to avoid getting bacteria into the wound. Throw the swab away after use.

Warning: External use only. Single use only. Never use if the package is damaged. Check the integrity of the individual sachet before use. In case of repeated discomfort, ask your doctor for advice. If swallowed, seek medical attention immediately and show the packaging. Use before the expiry date on the sachet. Store in a cool, dry place. After use, do not dispose of the compress in the environment.

Note! Instead of wipes you can use soap and water or an antibacterial solution. Soap and water is recommended.

8x Adhesive Bandage

Intended use: For use on minor cuts and abrasions. Protects minor wounds. Read instructions prior to use; retain for future reference.

Instruction for use: Clean wound and surrounding area and dry the skin before applying the plaster. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove plaster. Not suitable for children under 2 years old. In rare cases, a reaction to the adhesive bandage may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in the face. In both cases seek medical attention immediately. If you are diabetic consult your doctor before use.

2x Rolled Gauze Bandage

Intended use: To hold compresses or other medical dressings in place.

Instruction for use: Wrap the bandage around the area where

compress/dressing placed. Do not apply excessive pressure. Place the end under the lower layers or use a safety pin.

Warning: Never apply the bandage directly onto injured skin. Avoid pressing too hard. In case of discomfort, release the bandage to reduce the pressure. Store in a cool, dry place.

1x Sterile wound Dressing

Intended use: Bandage with compress for protecting a clean and dry wound.

Instruction of use: Cleaning and drying the wound and the skin around the wound. Apply the compress on the wound and wrap the bandage around. Finish by tying the two ends together

Warning: Replace the dressing on a daily basis, or more often if necessary. Single use only. Check the integrity of the individual sachet before use. Store in a cool, dry place. Use before the expiry date on the individual sachet. Must not get wet.

1x Burn gel

Intended use: For first aid use on burns, scalds and sunburn. First aid emergency burn gel. Cools, soothes and relieves pain.

Note! Water is best on burn wounds, continue to rinse with water on burns. Burn gel should only to be used on burns if there is no water available.

Instructions for use: Tear open sachet Apply gel liberally to affected area. Cover if necessary. Seek medical attention if pain persists.

Warning: Do not use on serious burns, mucous membranes or the eyes. For external use only. Discontinue use if irritation occurs. Do not store at extreme temperatures. Do not use if packaging has been opened or damaged.

1x Disposable Gloves

Intended use: To protect against infections. Disposable. Latex-free.

Warning: Non sterile. Avoid direct contact with the wound.

1x Scissors

Intended use: For cutting plaster, bandage, gauze or other small first aid devices. The scissors have a rounded cutting edge in order to protect the user from self inflicted injuries.

Warning: Keep out of reach of children.

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