Safety for life

OVERVIEW
FIRST AID KIT
WATERPROOF

6x Safety Pin

Intended use: For attachment of triangular bandages, first aid sling or other first aid bandages.

Warning: Keep out of reach of children. Be cautious when using this device because of its slightly pointed tip. Do not use if broken.

1x Scissors

Intended use: For cutting plaster, bandage, gauze or other small first aid devices. The scissors have a rounded cutting edge in order to protect the user from self inflicted injuries.

Warning: Keep out of reach of children.

1x Tweezer

Intended use: For removing splinters or other small objects in an emergency situation.

Warning: Keep out of reach of children. Be cautious when using this device because of its slightly pointed tip.

4x Disposable Gloves

Intended use: To protect against infections. Disposable. Latex-free.

Warning: Non sterile. Avoid direct contact with the wound.

1x Tick Remover

Intended use: For removal of ticks.

Instruction for use:

1. Press suppressed key
2. Create a clamp for the tick
3. Slowly rotate and remove tick

Warning: Follow instructions carefully. Make sure that the tick is completely removed. Watch out for any skin color changes and

seek medical attention if a red ring appears in the area where the tick was sitting.

1x Adhesive Tape

Intended use: For fixing the dressings, compress, cotton ball or other small medical pads.

Instruction for use: Apply the dressing / compress on clean and dry skin using the tape to keep it in place. After use, please handle it according to medical waste. Do not apply tape too tight, this will influence the blood circulation and feel uncomfortable.

Warning: Do not apply the tape directly on the wound. External use only, keep away from children. If wound bleeds excessively or skin becomes irritated or infected remove the tape and seek medical attention.

2x Blister Plaster

Intended use: Protects the skin and prevents blisters from forming. Reduces friction on skin. Providing instant pain relief and optimum healing conditions for heel blisters. Read instructions prior to use; retain for future reference.

Instruction for use: Clean and dry the skin properly before use. Remove the lower protective film without touching the adhesive surface. Apply the dressing directly on the skin and carefully smooth the edges and press firmly for 5-10 seconds on adhesive area to ensure adhesion. Remove the dressing once it begins to detach itself, gently peeling it off the skin. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Use immediately after opening protective wrapper. Do not use on wounds. If skin becomes irritated remove plaster and seek medical attention. If you are diabetic consult your doctor before use.

15x Trans. Pl in polybag

Intended use: For use on minor cuts and abrasions. Protects minor wounds. Read instructions prior to use; retain for future reference.

Instruction for use: Clean wound and surrounding area and dry skin before applying the adhesive bandage. Remove adhesive bandage from protective wrapper. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Use immediately after opening protective wrapper. Wrapper is sealed with natural rubber latex. Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove adhesive bandage. Not suitable for children under 2 years old. In rare cases, a reaction to the adhesive bandage may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in the face. In both cases seek medical attention immediately. If you are diabetic consult your doctor before use.

30x Plaster

Intended use: For use on minor cuts and abrasions. Protects minor wounds. Read instructions prior to use; retain for future reference.

Instruction for use: Clean wound and surrounding area and dry skin before applying the adhesive bandage. Remove adhesive bandage from protective wrapper. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Use immediately after opening protective wrapper. Wrapper is sealed with natural rubber latex. Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove adhesive bandage. Not suitable for children under 2 years old. In rare cases, a reaction to the adhesive bandage may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in the face. In both cases seek medical attention immediately. If you are diabetic consult your doctor before use.

10x PU-plaster

Intended use: Transparent PU plaster, composed of a pad on coated polyurethane film with acrylate adhesive and release top liner. For use on minor wounds or on IV catheter. Protects the wound from contamination, allowing the skin to breathe for enhanced healing. Read instructions prior to use; retain for future reference.

Instruction for use: Clean wound and surrounding area and dry skin before applying the plaster. Remove plaster from protective wrapper. Position pad over wound and press firmly on adhesive area to ensure adhesion. Then remove the release liner on the PU film. Change dressing every 24 hours or when wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Use immediately after opening protective wrapper. Wrapper is sealed with natural rubber latex. Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove plaster. Not suitable for children under 2 years old. In rare cases, a reaction to the plaster may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in

the face. In both cases seek medical attention. If you are diabetic consult your doctor before use.

3x Towel tabs

Intended use: Used for cleaning hands and face.

Instruction for use: Dip into water and let the towel absorb water, then squeeze it to dry and apply to hands or face for cleaning.

Warning: Do not contact with wounds. Dispose after use. Do not swallow.


1x Triangular Bandage

Intended Use: To immobilise and keep splints in place. Use this bandage as a sling to support an injured, sprained or broken arm.

Instruction for use: Unfold the bandage and attach around the neck as shown.

Warning: Product to be used once only. Use before the expiry date on the individual item. Check the integrity of the individual item before use.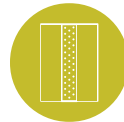

8x Adhesive Bandage

Intended use: For use on minor cuts and abrasions. Protects minor wounds. Read instructions prior to use; retain for future reference.

Instruction for use: Clean wound and surrounding area and dry the skin before applying the plaster. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove plaster. Not suitable for children under 2 years old. In rare cases, a reaction to the adhesive bandage may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in the face. In both cases seek medical attention immediately. If you are diabetic consult your doctor before use.

2x Sterile wound Dressing

Intended use: Bandage with compress for protecting a clean and dry wound.

Instruction of use: Cleaning and drying the wound and the skin around the wound. Apply the compress on the wound and wrap the bandage around. Finish by tying the two ends together

Warning: Replace the dressing on a daily basis, or more often if necessary. Single use only. Check the integrity of the individual sachet before use. Store in a cool, dry place. Use before the expiry date on the individual sachet. Must not get wet.

2x Rolled Gauze Bandage

Intended use: To hold compresses or other medical dressings in place.

Instruction for use: Wrap the bandage around the area where compress/dressing placed. Do not apply excessive pressure. Place the end under the lower layers or use a safety pin.

Warning: Never apply the bandage directly onto injured skin. Avoid pressing too hard. In case of discomfort, release the bandage to reduce the pressure. Store in a cool, dry place.

1x Respiratory sheet

Intended use: For emergency artificial respiration. To prevent direct contact with patient's mouth, nose and face. Provides rescuer protection in an emergency situation. Resists water and saliva allowing air to ventilate.

Instruction for use: Lift the chin forward and push back the forehead. Place the mask with (+)side up and (o) side on to the injured person's mouth. Pinch the injured person's nostrils with your thumb and your fore-finger. Take a deep breath and blow through the breathing mask(+) and check that the chest rises. After each respiration lift up your head and allow the air to expel from the injured person's lungs. Repeat blowing air into the injured person's mouth. Check the chest (12-15) respirations per minute. Note: If the chest doesn't rise, reposition the head (lift chin and push forehead back). Check the injured person's mouth to ensure free airway. Pinch the injured person's nostrils and try again.

Warning: Small children: Do not use the breathing mask on small children. Blow air through the nose and mouth at the same time 15- 20 respirations per minute. Single use only.

1x First Aid Blanket

Intended use: To protect against heat and cold. The emergency blanket can provide protection against heat, cold and humidity in emergency situations.

Instructions for use: Emergency blanket to be used once only. Open the sachet and unfold the blanket as required.

Two sides with different functions: The golden side absorbs heat and keeps you warm: providing a defence against potential hypothermia, maintaining your body temperature, and protecting you from the wind and humidity.

The silver reflective outer side keeps you cool: keeps you chilled (cars, tents, etc.).

Warning: Do not cover your head, danger of suffocating. Single use only. Do not use the emergency blanket during storms, near to incandescent substances or if using an electrical surgical device (e.g. DSA). Do not cut or create wholes. Store in a dry place.

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